

NuEyes e2 Mira Cast Solution

***** Please note you must have a smart TV as well as an internet/Wi-Fi connection.**

***** Your Smart TV must be connected to the same Wi-Fi network as your NuEyes e2 Glasses.**


- 1. While in the NuEyes e2 App hold down menu button for 2 Seconds until the admin menu appears.**




2. Scroll down to settings and press the middle button to enter.




3. Then select connected devices (press middle button)

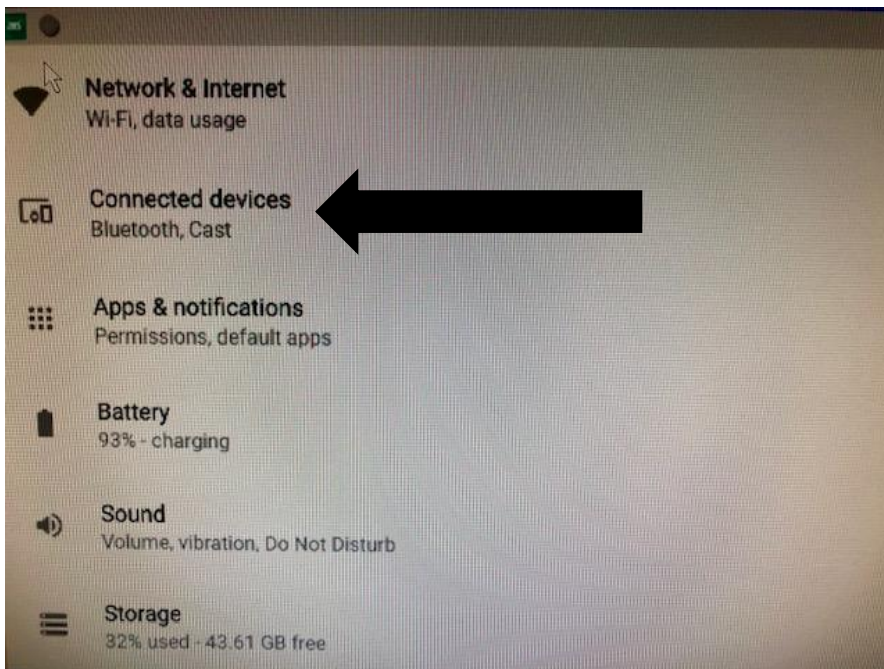


4. Now select Cast by pressing the middle button

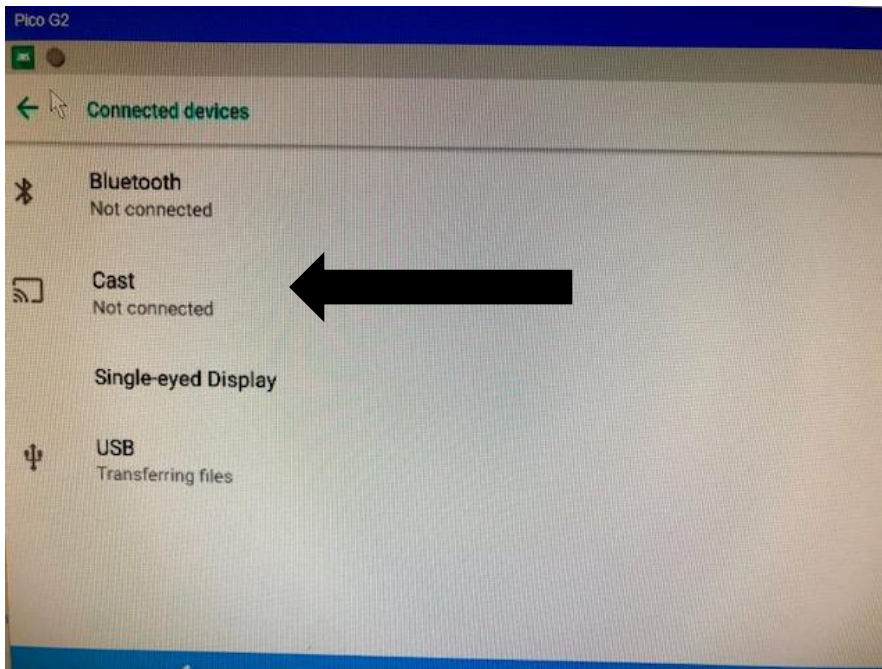

5. Make sure on the top right-hand side that enable wireless display is checked.

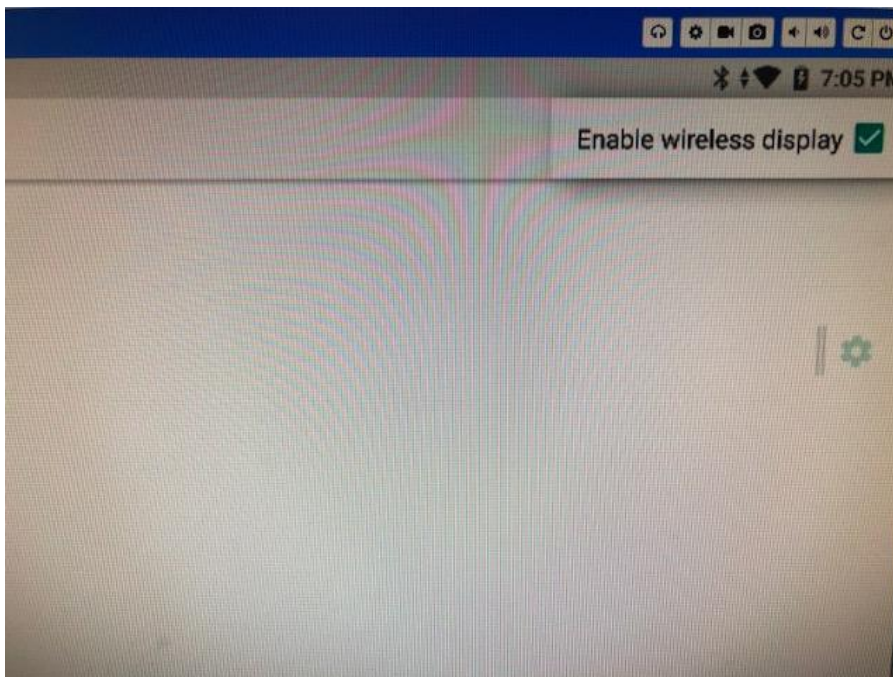

6. You will now see the device or TV you can connect to. Again, press the middle button to select the desired screen and the device will connect to it.

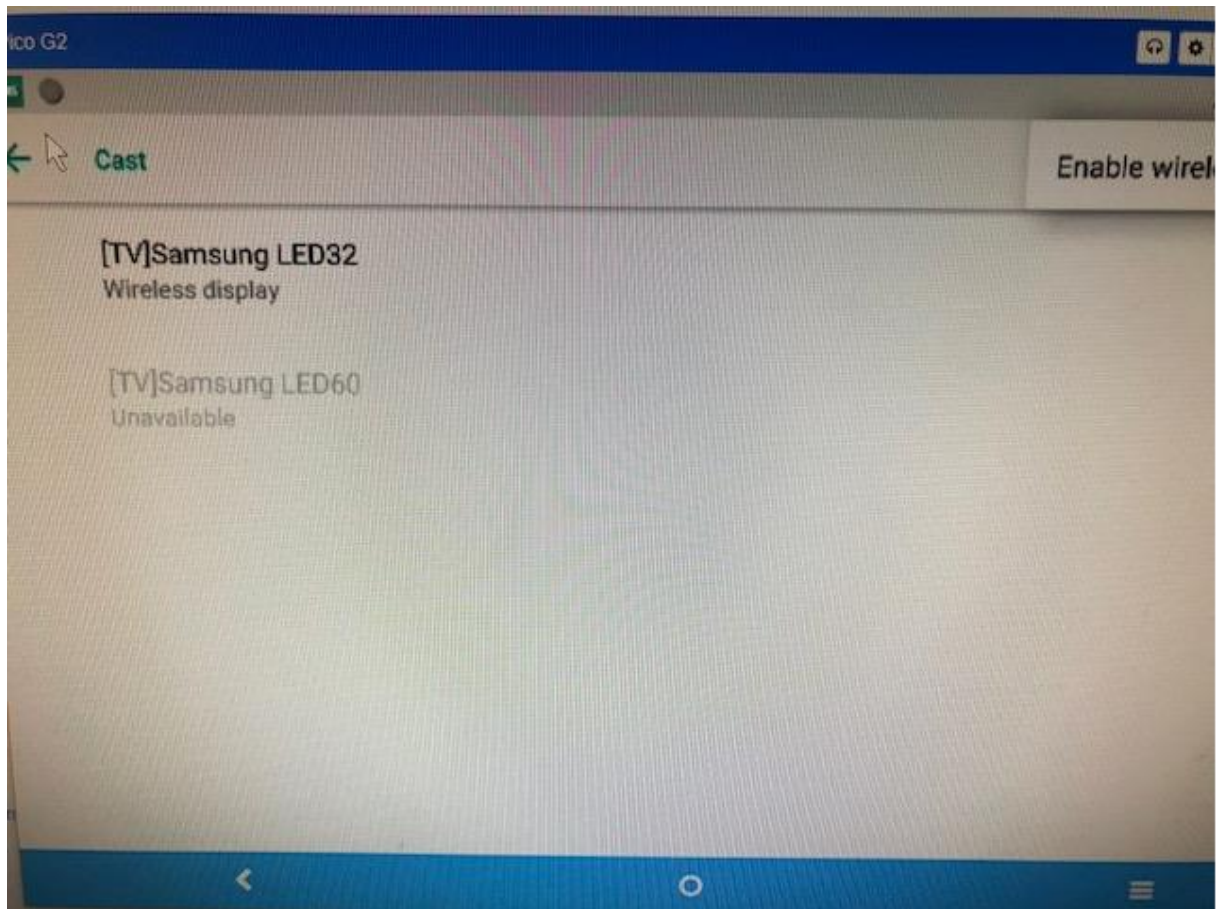

*****Please note the following*****

- **Mira Cast clarity depends on how fast your Wi-Fi signal is and how close the glasses are to the television.**
- **Fast movements of the glasses will distort the Mira Cast signal.**