



How to connect to WiFi (have a loved one, friend or family member assist you)

- Turn device on (Quick press, Power button)
- Wait 10 seconds for NuEyes application to upload
- Press the middle/menu button to access the Menu.
- Press the first/ mag button to cycle down to **SETTINGS**
- Press middle/ menu button (white screen will appear)
- Move the cursor by moving your head to the top **Network & Internet**
- Press Middle/menu button to select WiFi (In black text) find your home WiFi
- Type in your password- each character press Middle/menu button
- Press Middle/menu button select the green check **WiFi connected**