



How to connect/pair remote/controller (have a loved one, friend or family member assist you)

### **Remote controller is optional**

- Have remote/controller ready
- Turn device on (2 second press, Power button), you will see NuEyes logo with sound
- Screen will read (loading...)
- **Please press the home button (Third/indent Pico logo) on the remote/Controller**
- Screen will display remote/controller and prompt you...  
The controller is not detected. Please press the home button (Third/ Pico logo) on the remote/Controller
- Once remote is connected you will see cursor

(+, O- Teal circle with white line in screen)

- Hold remote up in front of e2 you should see virtual remote.
- If it is not aligned properly - press the Pico/third bottom button on remote.  
\*Remote/Controller Middle button can be used to Cycle up to 18x MAX or Cycle down to MIN 3x magnification