

## NuEyes Mira Cast Solution


**\*\*\* Please note you must have a smart TV as well as an internet/Wi-Fi connection.**

**\*\*\* Your Smart TV must be connected to the same Wi-Fi network as your NuEyes Glasses.**


- 1. Hold down menu button for 30+ Seconds until the admin menu appears.**



2. Scroll down using your finger controller/Bluetooth Remote to the tab that says “Wi-Fi”; Then click or depress controller to enter Wi-Fi mode.

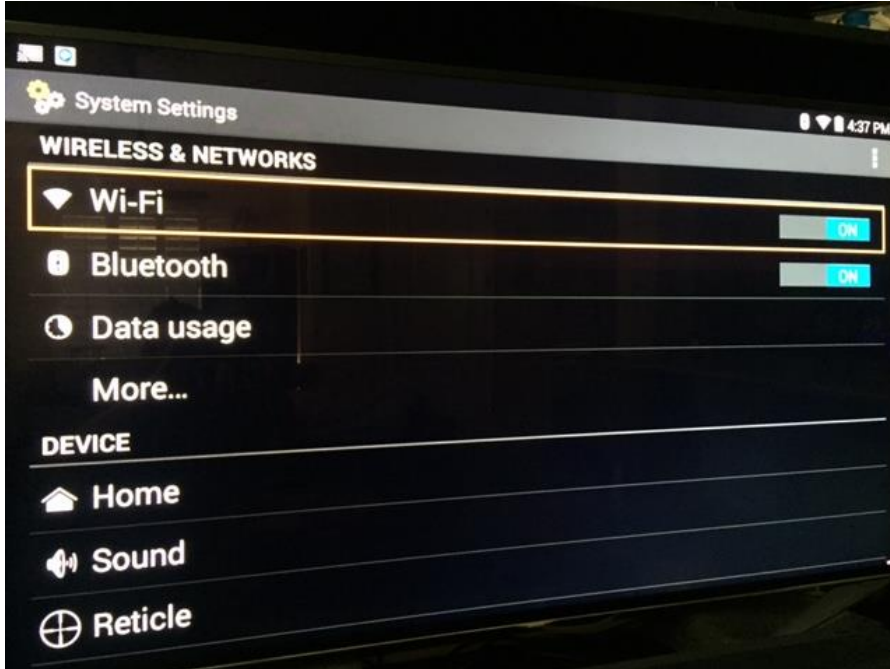

3. Select Appropriate Wi-Fi connection

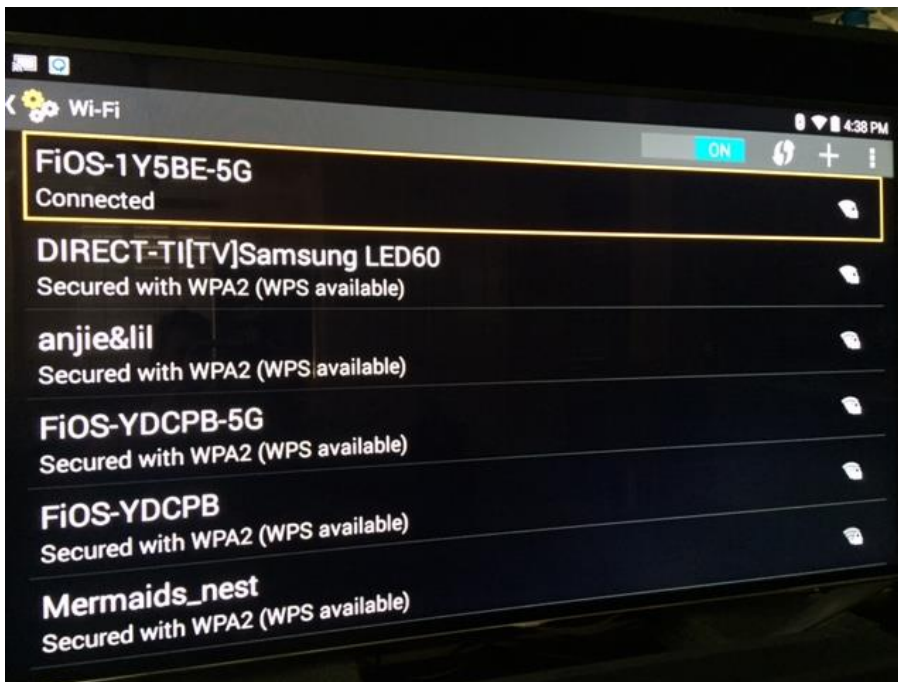


**4. Use on board key board to log in.**




**5. Press back button once (no hold) to Re-enter admin mode**



6. Scroll down using your finger controller/Bluetooth Remote to the tab that says “display”; Then click or depress controller to enter display mode.

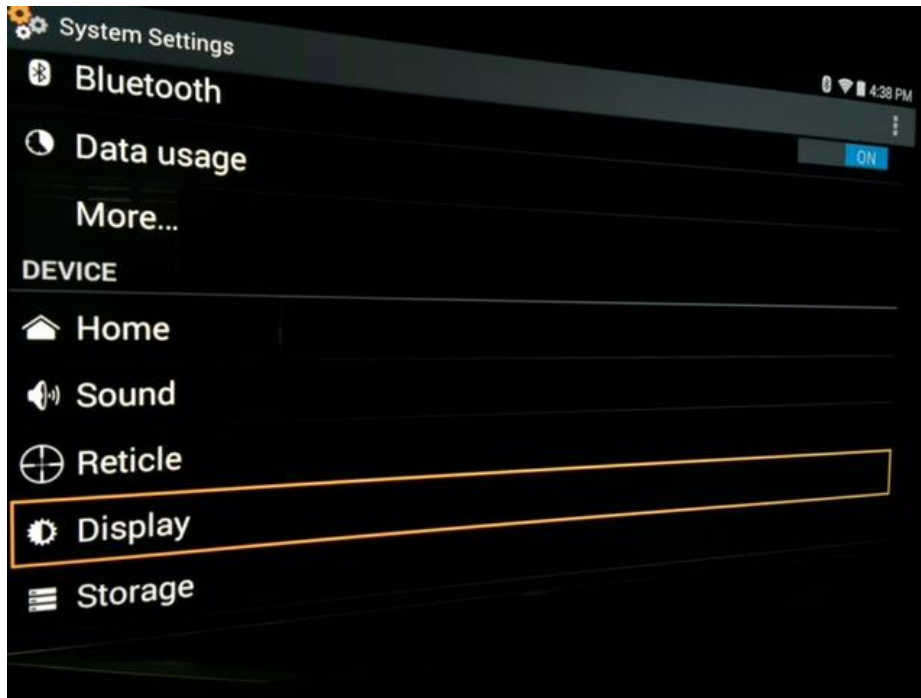

7. While in display mode scroll down to cast screen and depress or click on cast screen.

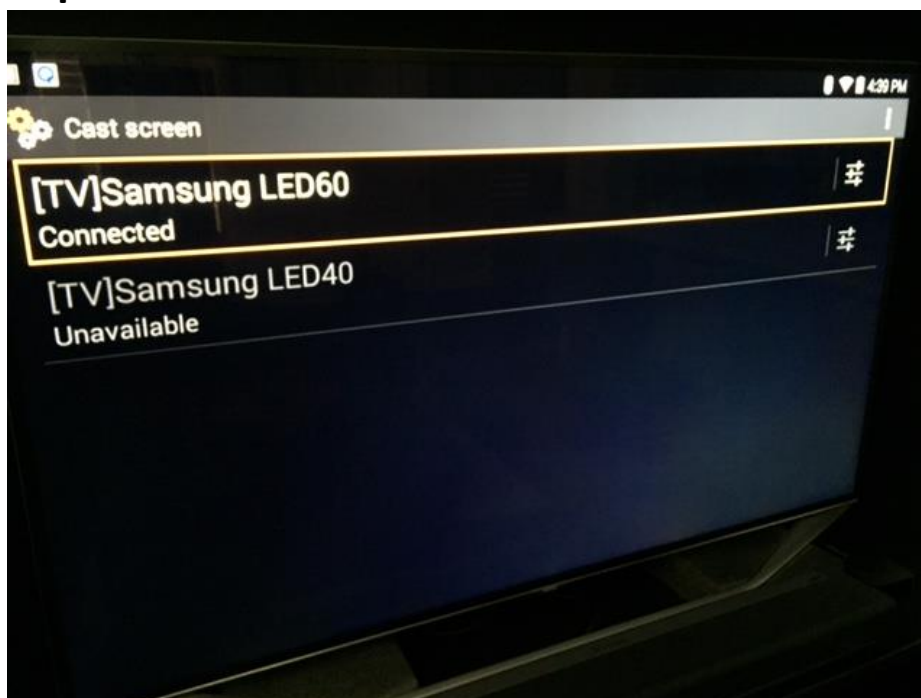

8. When in Cast screen mode press menu button and enable wireless display.



9. Once Enabled you will see your Smart TV connect to the NuEyes Glasses.



10. Once your NuEyes is connected to your smart TV please press the back button twice. Once to get out of the display menu, and once more to exit admin menu; enabling the NuEyes software.



**\*\*\*Please note the following\*\*\***

- Mira Cast clarity depends on how fast your Wi-Fi signal is and how close the glasses are to the television.
- Fast movements of the glasses will distort the Mira Cast signal.